# BUKU PANDUAN PENGGUNAAN

# NIVA

NON INVASIVE VASCULAR ANALYZER

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# NIVA

NON INVASIVE VASCULAR ANALYZER

Reg No	KEMENKES RI AKD XXXXXXXXX
Product Model	NIVA 1000
Input Voltage (Mains)	230 - 240VAC 50Hz
Power consumption	< 200 Watt
Output Voltage (DC Adapter)	14.4VDC 6A
Dimension (Control Unit)	385mm x 280mm x 137mm
Weight	5 kg
Operating Temperature	+10 to +40°C
Operating Humidity	15 - 85% (no condensation)
Atmospheric pressure	700 - 1060 hPa
Serial number	011222NV0001

⚠️ ⚡️ ♻️ ⚙️ ⚠️

**USE ONLY ORIGINAL CHARGER**  
**GUNAKAN HANYA CHARGER ORIGINAL**

**DESIGNED IN INDONESIA**  
**MADE IN INDONESIA** IPX0

PT Salama Citra Nusantara Perkasa Tbk. (SCNP)  
Dusun Padiwangi, RT 003 RW 004  
Cibeugis, Kab. Bogor, Indonesia

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### 3.1 UMUM

Nama Produk	: Non- Invasive Vascular Analyzer (NIVA)
Model Produk	: NIVA 1000
Medical Device Class	: Class B
Device Protection Class	: Class I – Type BF (NIBP)
Safety standards	: EN/IEC60601-1
Dust and water resistance	: Class IPX0
Dimensi trolley (L x P x T)	: 520 mm x 520 mm x 1380 mm
Dimensi control unit (L x P x T)	: 385 mm x 260 mm x 135 mm
Berat trolley (without printer)	: 60 kg
Konsumsi Daya	: < 200 Watt
Tampilan	: Touchscreen 23.8 inch Resolusi 1920 x 1280 mm
Metode Pengukuran	: Osilometrik
Area Pengukuran (tekanan darah, Sistolik)	: 60 – 250 mmHg
Area Pengukuran (tekanan darah, Diastolik)	: 40 – 220 mmHg
Akurasi	: $\pm 5$ mmHg
Ukuran Cuff yang digunakan	: Adult Size
Jumlah Sensor yang digunakan sewaktu pengoperasian	: 4x Cuff Manset Tekanan Darah 2 x Photoplethysmograph sensor
Tekanan Cuff Maksimum	: 300 mmHg

### 3.2 MAIN POWER SUPPLY

Jenis	: Uninterruptible Power Supply
Peringkat Daya	: AC 220-240 V
Frekuensi Daya	: 50 / 60 Hz
Konsumsi Daya	: 1,6 kVA

### 3.3 CONTROL UNIT POWER SUPPLY

Jenis	: Rechargeable Lithium-Ion Battery Pack
Cell Type	: LiFePO4 32700
Nominal Voltage	: 12.8 V
Nominal Capacity	: 12 Ah
Max Continuous Charge Current	: 6A
Max Continuous Discharge Current	: 12A
Dimension (LxWxH)	: 150 x 85 x 77 mm

Fitur BMS : Passive balancing, over charge protection, over discharge protection, over current protection, short protection.

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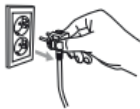
The screenshot shows a login interface for the NIVA system. It features a dark blue background with the ITS and SCNP logos at the top. The title 'NIVA (Non Invasive Vascular Analyzer)' is centered. Below the title are two white input fields for 'Username' and 'Password'. At the bottom, there is a white button with a green arrow and the text 'MASUK'.

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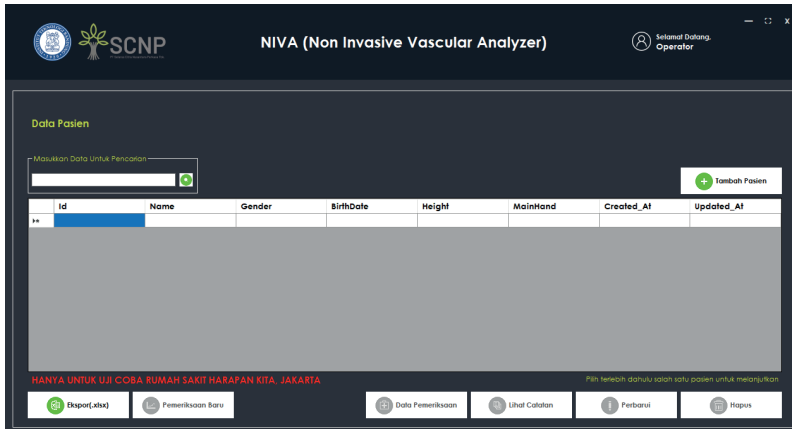
The screenshot shows the 'Tambah Pasien Baru' form in the NIVA software. The form has the following fields:

- ID Pasien: [Empty text box]
- Nama: [Empty text box]
- Jenis Kelamin: [Dropdown menu]
- Tanggal Lahir: 08 April 2022
- Tinggi Badan: [Empty text box]
- Tangan Dominan: [Dropdown menu]

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The screenshot shows the NIVA software interface. At the top, it says "NIVA (Non Invasive Vascular Analyzer)" and "Selamat Datang, Operator". Below this is a "Data Pasien" section with a search bar and a "Tambah Pasien" button. A table lists two patients:

Id	Name	Gender	MainHand	Created_At	Updated_At
ID0001	Pasien 1	Laki-laki	Kanan	07/04/2022 7:41	07/04/2022 7:12
ID0002	Pasien 2	Laki-laki	Kanan	08/04/2022 14:31	08/04/2022 14:31

A "Patient Deletion" dialog box is open, asking "Are you sure you want to delete this patient: ID0001 1" with "Yes" and "No" buttons. At the bottom, there are buttons for "Ekspor(.xlsx)", "Pemeriksaan Baru", "Data Pemeriksaan", "Lihat Catatan", "Perbarui", and "Hapus". A red warning message at the bottom reads: "HANYA UNTUK UJI COBA RUMAH SAKIT HARAPAN KITA, JAKARTA".

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The screenshot shows the NIVA software interface after the deletion. The "Data Pasien" section now only contains one patient in the table:

Id	Name	Gender	BirthDate	Height	MainHand	Created_At	Updated_At
ID0002	Pasien 2	Laki-laki	12/01/1983 14:30	174	Kanan	08/04/2022 14:31	08/04/2022 14:31

The "Patient Deletion" dialog box is no longer present. The rest of the interface, including the search bar, "Tambah Pasien" button, and bottom navigation buttons, remains the same. The red warning message is also present.

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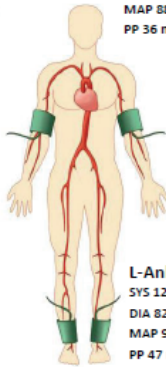
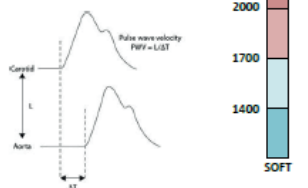
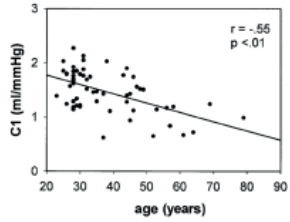
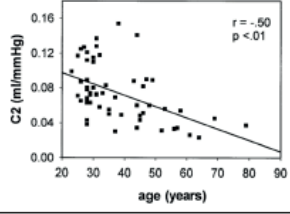
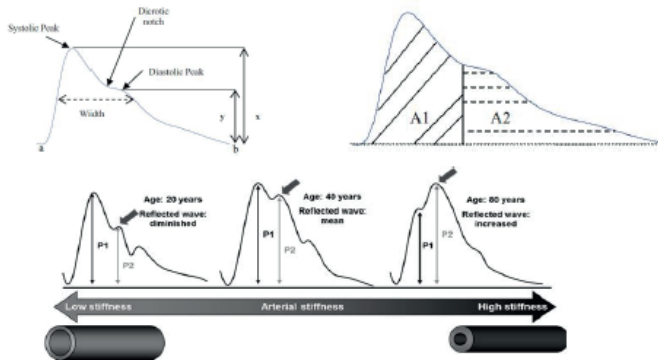
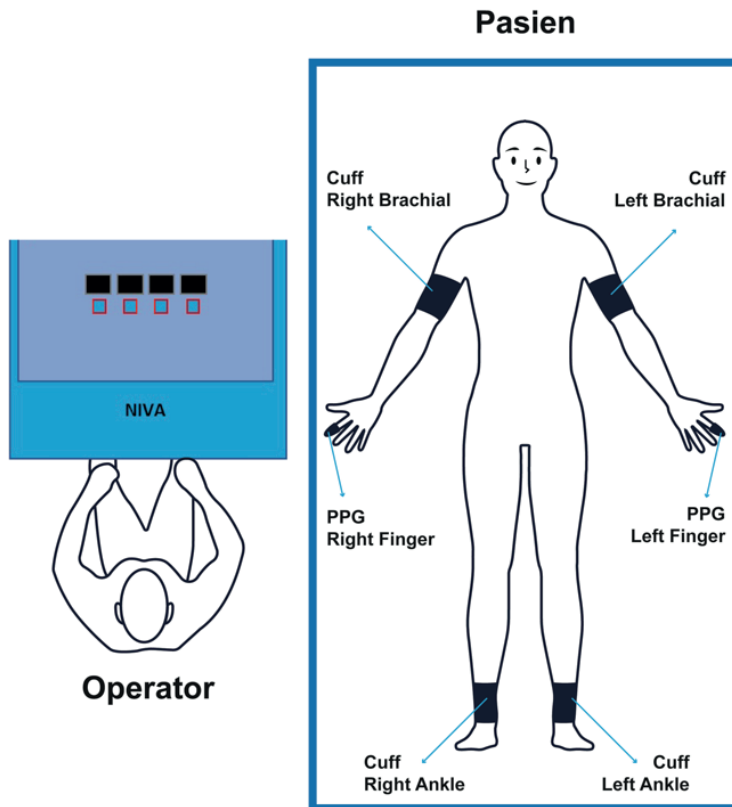
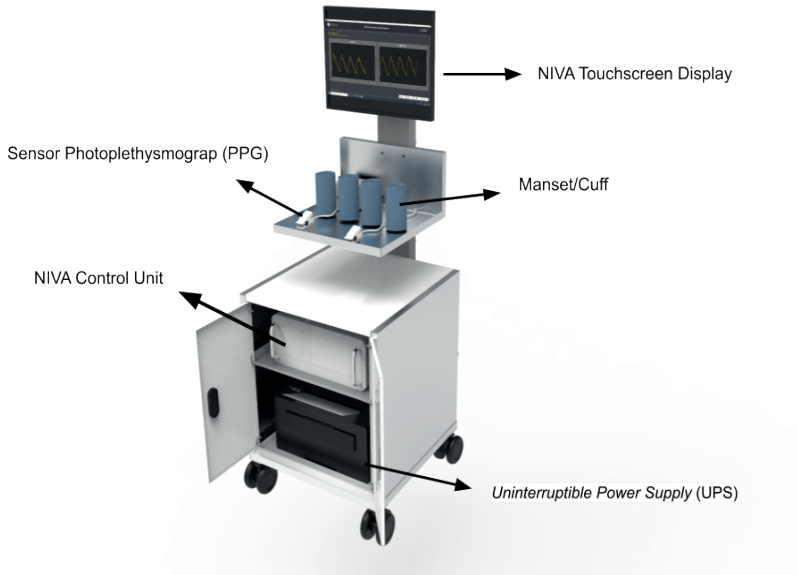
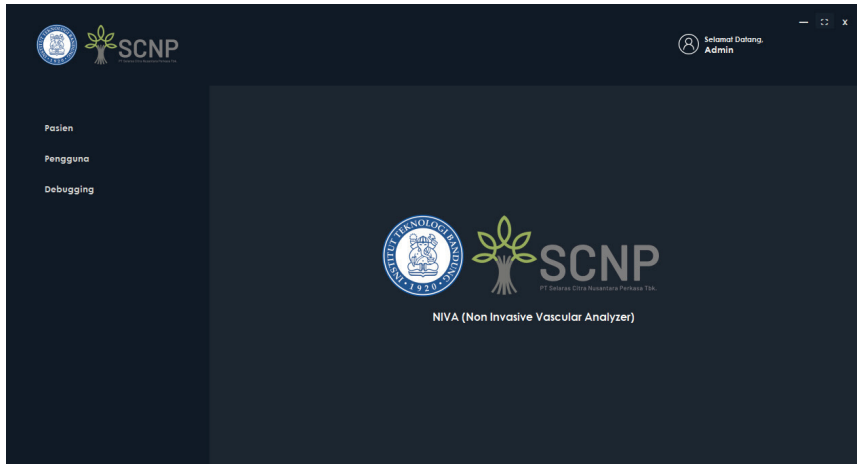
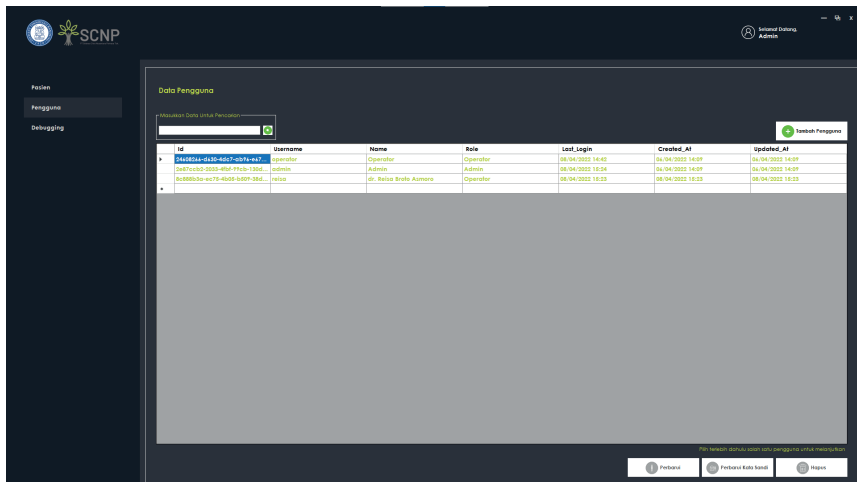
# NIVA

NON INVASIVE VASCULAR ANALYZER

For latest **Manual Book**,  
please download from this link

<https://sharefile.sncp.co.id/niva>

SCAN ME

## PRELIMINARY

### 1.1 Purpose of Document Creation

The Non-Invasive Vascular Analyzer (NIVA) application manual document was created for the following purposes:

1. Describe and explain the NIVA application to related users.
2. As a guide to the configuration and use of the NIVA application.

Interested parties use the NIVA application to retrieve data from patients by attaching a cuff/cuff and Photoplethysmograph (PPG) sensor found on the NIVA device to the patient and have the right to use this document, hereinafter referred to as the Operator.

### 1.2 System General Description

#### 1.2.1 General Description of the Application

The general description of the application includes an explanation and general description of the NIVA application developed, the main functions of the NIVA application that will be provided to users and user characteristics.

#### 1.2.2 General Description of Application Requirements

A general description of the requirements for the NIVA application that will be implemented includes all technical information that becomes a reference in implementing, using, and running the NIVA application.

### 1.3 Document Description (Overview)

This document was created to provide a guide for using the NIVA application which contains the following information:

1. Introduction: Contains general information which is the introductory part including the purpose of making the document, general description of the system and document description.
2. General Information: Contains descriptions of Symbols, Fair use of the appliance, Safety instructions, and Instructions for product labels and production codes.
3. Device Specifications: Contains device information used for NIVA applications including hardware and software.
4. Instructions for Use of NIVA: Contains information on how to use and operate the NIVA application for operators.

## GENERAL INFORMATION

### 2.1 Symbol Description

	Attention Watch out		Read Guidebook
	Alternating current and direct current		Class I Tools (Protective earth)
	Disposal Instructions		BF type
	Serial Number		Producer
	Handle with care		Orientation instructions Top direction
	Keep away from rain water		Glassware
	Do not slam		Do not be stacked
	Don't be stepped on		Read Guidebook

## 2.2 Fair use (intended use)

Medical purpose	:	It is a non-invasive tool that helps in the diagnosis and detection of cardiovascular diseases.
User population	:	Legal and certified medical personnel, such as doctors and nurses.
Patient population	:	Only for adults over 18 years.
Usage environment	:	This tool is for use in vascular laboratories, clinics, hospitals, doctors' offices and other health facilities where needed.
Usage period	:	Minimum 1 year provided that the maintenance steps have been carried out correctly from the date of production.
Measurement parameters	:	Brachial and ankle blood pressure through the cuff Finger blood pressure via photoplethysmography sensor
Calculation parameters	:	<ul style="list-style-type: none"><li>❖ Systolic Pressures</li><li>❖ Diastolic Pressures</li><li>❖ ABI (Ankle Brachial Index)</li><li>❖ baPWV (brachial – ankle Pulse Wave Velocity)</li><li>❖ Compliance large arteries (C1)</li><li>❖ Compliance small artery (C2)</li><li>❖ Pulse Width</li><li>❖ Pulse Interval</li><li>❖ Augmentation Index</li><li>❖ Inflection Point Area</li><li>❖ Stiffness Index</li><li>❖ Delay Time</li><li>❖ Systolic Amplitude</li><li>❖ Estimated vascular age</li><li>❖ Heart Rate</li></ul>
For attention	:	Recommendations and matters relating to safety described in the manual must be observed.

### 2.3 Safety instructions

The following are things that you should pay attention to to ensure safety when using the device and prevent damage and injury to yourself and others, such as death, serious injury, minor injury, to physical damage.

	Do not use a torn or damaged power cord or plug. Otherwise, it may result in electric shock, short circuit, or fire.
	Do not touch the power plug with wet hands. Otherwise, electric shock or injury may occur
	Be sure to plug the three-prong power plug into an outlet that has been grounded for medical use. Otherwise, electric shock or leakage current may occur. Use the adapter provided.
	Do not use a branched outlet with any other appliance. Otherwise, electric shock or leakage current may occur.
	Do not puncture or scratch buttons or layers with sharp or pointed objects. Incorrect diagnosis and accident or equipment damage may occur.
	Do not connect the mains plug to the wall outlet if the mains voltage is outside the following ranges: 220 – 240 VAC, 50/60Hz. This can cause a fire.
	This device is for use only by qualified medical personnel, or under the guidance of such personnel. Otherwise wrong diagnosis or tool failure may occur.
	The measurement results should only be interpreted by a doctor. If you are concerned about the measurement results, consult your doctor. Otherwise, incorrect diagnosis and treatment may occur.
	This device is intended to take measurements for inspection. Do not use the device for emergency patients. Otherwise, accidents may occur.
	If the pressure does not stop during measurement or other abnormal conditions occur, immediately press the Power (OFF) button to turn off the instrument and remove the cuff or air tube from the patient's body. Otherwise, the peripheral nerves can be damaged.
	Use only original and specified supplies of tools or accessories, such as UPS, cables, cuffs, power adapters, and other components. Otherwise, accidents or equipment damage may occur.

	Do not install any software other than those specified by the manufacturer. Otherwise, performance degradation and equipment failure may occur.
	Do not connect the air hose or cuff to the tube of any other body-mounted device. Otherwise, air may enter the veins and accidents may occur.
	Do not attach arm cuffs to patients during medical treatment with intravenous drips or blood transfusions. This can lead to incorrect diagnosis and treatment.
	Do not use this device in patients whose data does not express pain. If the cuff pressure does not stop within a certain time, internal bleeding can occur above the arm or ankle (ankle).
	This device cannot be used during MRI imaging. This can cause an accident.
	Observe the patient's condition constantly during the measurement to avoid possible accidents.
	Do not take measurements without cleaning the device, cuff and sensors, to avoid possible infection.
	Do not use this device in patients with pacemakers. This can cause an accident.
	Do not use this device near flammable gases, such as anesthetic gas or high-pressure oxygen cylinders. Fire and explosion may occur.
	Do not place the device in a location where there is a possibility of splashing water or other chemicals on the appliance. This may result in electric shock and equipment damage.
	Do not connect other electronic devices other than those specified by the manufacturer. For Printers, only use printers that have been specified by the manufacturer.
	Use a dry or damp cloth to clean the appliance. Do not allow water to enter the appliance. This may cause electric shock and equipment damage.

	<p>Pay attention to the following points when handling the cord plug (main cable). Carelessness in observing this can result in electric shock and equipment damage:</p> <ul style="list-style-type: none"> <li>❖ Do not damage the cord</li> <li>❖ Do not cut the cord</li> <li>❖ Do not change the cord</li> <li>❖ Do not pull the cord with unnatural force</li> <li>❖ Don't twist the cord</li> <li>❖ Do not fold the cable too tightly when using the device</li> <li>❖ Do not place heavy objects on the cord</li> <li>❖ Don't let the cord get squeezed</li> </ul>
	<p>Insert the power cord fully into the plug. Otherwise, it may result in electric shock, short circuit, or fire.</p>
	<p>Remove dust from the power cord. Otherwise, it may result in electric shock, short circuit, or fire.</p>
	<p>When finished using the appliance, turn off the appliance by pressing the Power (OFF) button, turn off the PC layer (shutdown), and turn off the UPS by pressing the power button on the UPS. Otherwise, it may result in equipment damage, electric shock, short circuit, or fire.</p>
	<p>When removing the power cord from the plug, grasp and pull the plug, not the cord. Pulling on the cord can damage the inside of the cord and cause a short circuit and electric shock.</p>
	<p>If there is a power failure while using the appliance, turn off the appliance by pressing the Power (OFF) button and turn off the UPS by pressing the Power button on the UPS. Failure to do this can lead to internal bleeding in the patient, equipment damage and other problems.</p>
	<p>If the appliance gets wet, immediately turn off the appliance, layer PC, and UPS. Immediately wipe with a cloth until completely dry. Otherwise, it may result in electric shock, short circuit, or fire.</p>
	<p>Do not take measurements in patients with the following conditions. Otherwise, misdiagnosis and treatment may occur.</p> <ul style="list-style-type: none"> <li>❖ Patients with aneurysms</li> <li>❖ Patients with acute peripheral circulation problems.</li> <li>❖ Patients with irregular pulse.</li> </ul>
	<p>At the time of measurement, external vibrations and patient movements can cause artifacts in the signal so that the results become less accurate or an error occurs in the measurement.</p>
	<p>If there is acute inflammation, pyogenic ailment, or external injury to the area where the cuff is attached, follow the doctor's directions. Symptoms may become more severe.</p>
	<p>If there are abnormal results or there are doubts about the measurement results, please verify the patient's condition first. The patient's condition may have exceeded the instrument's measurement limit. Also verify the</p>

	installation condition of PPG cuffs and sensors, and hoses so that they are not squeezed, leaked or clogged.
	Do not use the device in very noisy locations or close to an MRI, CT, X-ray room, operating room, or other areas where the radio signal is very strong. This can lead to incorrect diagnosis and treatment.
	Place the cuff directly on the patient's skin (bare skin) or a thin cloth. Otherwise, misdiagnoses and errors may occur.
	If the air hose, cuff, or PPG sensor is damaged, replace it with a new one. For purchases, you can contact the service center of PT. SCNP.
	Do not use tools in moving vehicles such as ambulances. This can lead to incorrect diagnosis and treatment.
	Do not place heavy objects on the appliance. This can cause permanent damage to the tool.
	Do not disassemble, repair or modify the appliance. This may cause electric shock and equipment damage.
	Before cleaning the appliance, make sure the power cord is unplugged. Otherwise, it may cause electric shock and injury.
	The battery charging current must be lower than the max charge current written in the specifications. Otherwise, it may cause damage to the battery.
	The discharging current of the battery must be lower than the max discharge current stated in the specifications. Otherwise, it may cause damage to the battery.
	The battery cell can be in an over-discharged state due to the characteristics of the self-discharge cell if it is not used for a long time. To prevent over-discharging, the battery must be charged regularly (recommendation every 3 months). Over discharge can cause a decrease in cell performance and damage to the battery.

## 2.4 Safety instructions regarding waste (disposal)

This mark is indicated on the product and its literature, and indicates that the product should not be disposed of with other household waste at the end of its useful life.

To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of waste and recycle it responsibly to promote the sustainable reuse of material resources.

## 2.5 Instructions for product labels and production codes

The following shows the location of the product serial number which is located on the product label. You can find the product label affixed to the back of the tool.

011222NV0001

<p>2 digit Batch Number →</p> <p>4 digit month and year of production MMY →</p>	<p>→ 4 digit product serial number</p> <p>Product Code            Nv = Niva            Sy = Syringe Pump            If = Infusion Pump            Vn = Ventilator            PM = Patient Monitor</p>
---------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

# NIVA

NON INVASIVE VASCULAR ANALYZER

Reg No	KEMENKES RI AND XXXXXXXXX
Product Model	NIVA.1000
Input Voltage (Main)	220 - 240VAC 50Hz
Power consumption	< 200 Watt
Output Voltage (DC Adapter)	14.4VDC 6A
Dimension (Control Unit)	382mm x 260mm x 137mm
Weight	3.7kg
Operating Temperature	+10 to +40°C
Operating Humidity	15 - 85% (no condensation)
Atmospheric pressure	760 - 1060 hPa
Serial number	011222NV0001

SN 011222NV0001

DESIGNED IN INDONESIA  
MADE IN INDONESIA

IPX0

**USE ONLY ORIGINAL CHARGER  
GUNAKAN HANYA CHARGER ORIGINAL.**

PT. Sistem Citra Nustara Perkasa Tbk. (SCNP)  
 Duam Paerang, RT 003 RW 004  
 Cileungsi, Kab. Bogor, Indonesia

## DEVICE SPECIFICATIONS

### 3.1 GENERAL

Product name	: Non-Invasive Vascular Analyzer (NIVA)
Product Model	: NIVA 1000
Medical Device Class	: Class B
Device Protection Class	: Class I – Type BF (NIBP)
Safety standards	: EN/IEC60601-1
Dust and water resistance	: IPX0 class
Trolley dimensions (W x W x H)	: 520mm x 520mm x 1380mm
Dimensions of control unit (W x W x H)	: 385mm x 260mm x 135mm
Trolley weight (without printer)	: 60 kg
Power Consumption	: < 200 Watt
Appearance	: Touchscreen 23.8 inches Resolution 1920 x 1280 mm
Measurement Method	: Oscilometric
Measurement Area(blood pressure, Systolic)	: 60 – 250 mmHg
Measurement Area(blood pressure, Diastolic)	: 40 – 220 mmHg
Accuracy	: ±5 mmHg
Cuff size used	: Adult Size
Number of Sensors used during operation	: 4x Blood Pressure Cuff Cuff 2 x Photoplethysmograph sensors
Maximum Cuff Pressure	: 300 mmHg

### 3.2 MAIN POWER SUPPLY

Type	: Uninterruptible Power Supply
Power Rating	: AC 220-240 V
Power Frequency	: 50/60 Hz
Power Consumption	: 1.6 kVA

### 3.3 CONTROL OF POWER SUPPLY UNITS

Type	: Rechargeable Lithium-Ion Battery Pack
Cell Type	: LiFePO4 32700
Nominal Voltage	: 12.8 V
Nominal Capacity	: 12 Ah
Max Continuous Charge Current	: 6A

Max Continuous Discharge Current	: 12A
Dimensions (LxWxH)	: 150 x 85 x 77 mm
BMS Features	: Passive balancing, over charge protection, over discharge protection, over current protection, short protection.

### 3.4 ENVIRONMENTAL CONDITIONS

Temperature Range	: +10 to+40°C
Humidity Range	: 15 – 85% (no condensation)
Atmospheric Pressure	: 700 – 1060 hPa

### 3.5 TRANSPORTATION AND STORAGE CONDITIONS

Temperature Range	: +10 to+60°C
Humidity Range	: 10 – 95% (no condensation)
Atmospheric Pressure	: 500 – 1060 hPa

### 3.6 DISPLAY SCREEN

Screen Diagonal	: 23.8 Inch
Screen Resolution	: 1920 x 1080 Full HD
Aspect Ratio	: 16:9
Touch screen	: Yes
Viewing angle	: ±89° Horizontal and Vertical
Contrast Ratio	: 1000:1
Panel Type	: IPS Anti-glare 250 nits

## NIVA INSTRUCTIONS

### 4.1 NIVA App Users

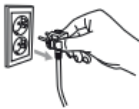
Operators who will use the NIVA application at least have basic knowledge and understanding of computer interfaces and procedures for installing cuffs/cuffs in the blood pressure data collection process.

### 4.2 Introduction and Training

Human resources who will be involved in the operational use of the NIVA application in their daily life must first be given adequate introduction and training so that they can operate the NIVA application properly so that field obstacles, especially Operational Human Errors can be prevented.

### 4.3 Installation of NIVA support devices

Before operating the NIVA, there are several things that must be considered and ensured by the operator so that the NIVA can operate properly, including:

Connect the NIVA power supply Plug to a 220 V AC power source.

The condition of the NIVA device is ON by pressing the power button.


The Charge button remains ON.

NIVA is equipped with a Battery Management System (BMS) as a safety device feature.

#### 4.4 Login Page

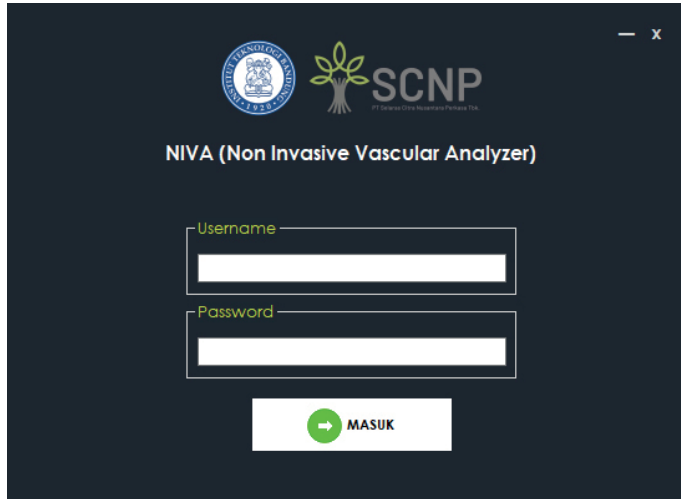After performing all the steps contained in the installation section of the NIVA support device, you will then be directed to the login page to be able to enter the NIVA system. The login page will look like the following image:

Then it is expected to enter the username and password that have been given / created.  
Example (Default)

Username: Operator


Password: user123

After that, the operator can continue using NIVA by clicking on the SIGN IN button. Then you will be directed to the NIVA main page.

## 4.5 Main Page

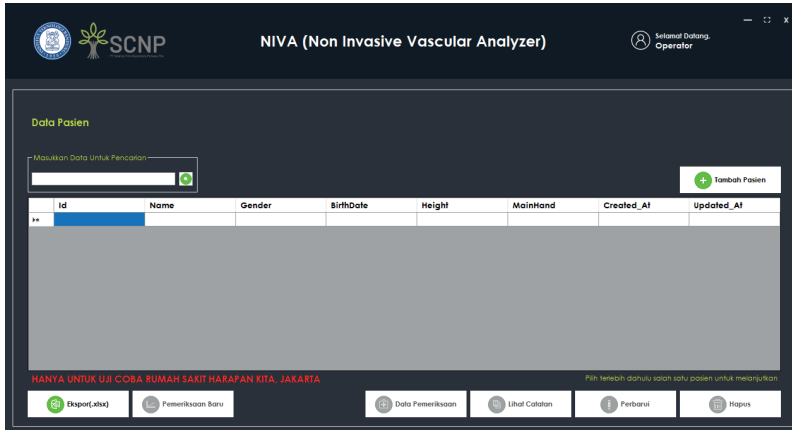After successfully performing the steps on the login page, you will be directed to the main page that looks like the following image:

On the main page view, there are several panels, buttons, and tables, each of which has its own function as follows:

- Data Input Panel for Search which functions to search patient data.
- Add Patient button to add data by inputting new patient data. Table of patient data already registered in the NIVA system.
- The Export button (.xlsx) is used to export patient measurement data into .xlsx format which can later be opened in Microsoft Excel.
- The New Examination button is used to perform a new examination on a patient who has been registered and will be active after selecting a patient to be examined.
- The Examination Data button is used to view the results of the patient's examination and will be active after selecting the patient for whom the results of the examination data will be seen.
- The View Record button is used to view the registered patient's special record and will be activated after selecting the patient whose special record will be viewed.
- The Update button is used to update registered patient data in the form of Name, Gender, Date of Birth, Dominant Hand, Height and will be active after selecting the patient whose data will be updated.
- The Delete button is used to delete patient data that has been registered and will be active after selecting the patient data to be deleted.

#### 4.6 Add New Patient

After clicking on the add patient button, the Add new patient page will appear as shown below:

The screenshot shows the 'Tambah Pasien Baru' form in the NIVA application. The form has the following fields:

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At the bottom right, there are two buttons: 'Kembali' and 'Selanjutnya'.

On the add new patient page there is a text box for patient ID, Name, Gender, Date of Birth, Height, and Dominant Hand which can be filled in according to the patient data to be examined.

After all the data is filled in then click on the next button it will be directed to a special notes page as shown below:


The screenshot shows the 'Tambah Pasien Baru' form in the NIVA application. The form has the following fields:

- Catatan Khusus

At the bottom right, there are two buttons: 'Kembali' and 'Selanjutnya'.


This page is used to provide special notes to patients who will be examined so that later they can be taken into consideration during the analysis process.

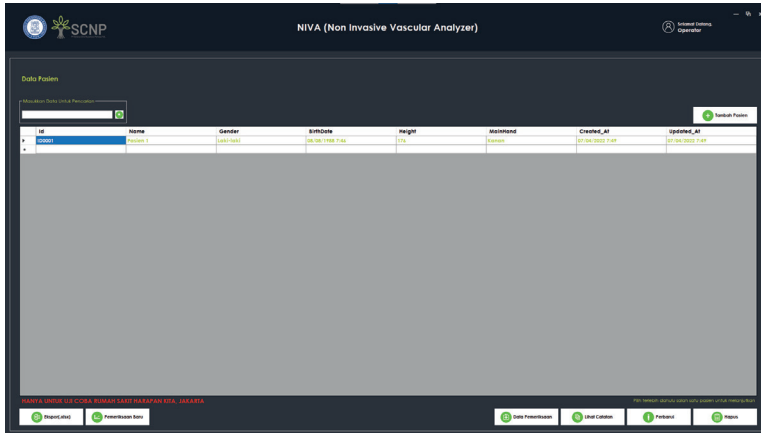After giving a special note, click the next button and you will be redirected back to the main page as shown below:


#### 4.7 New Checks

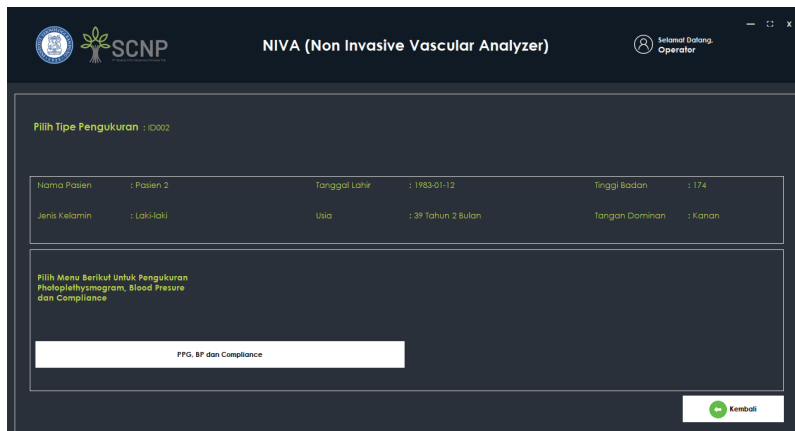
New examinations are carried out on patients who have been registered. After selecting the patient for a new examination then clicking on the new examination button will be directed to a new examination page as follows:

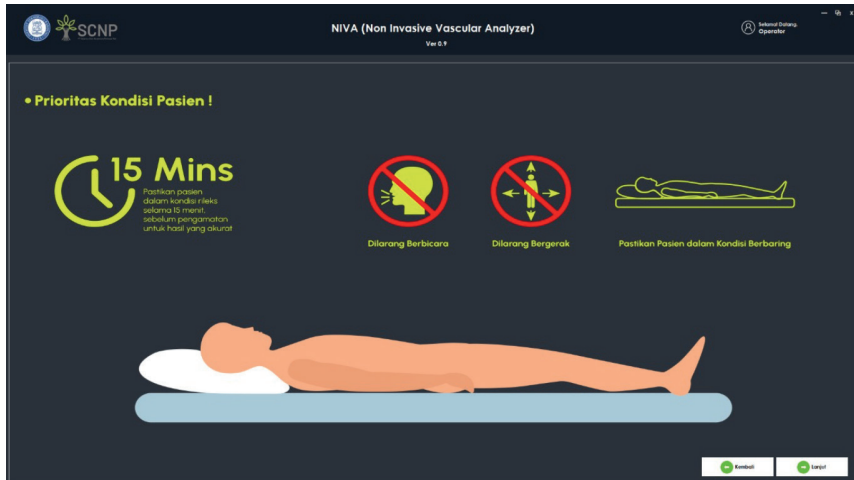
Then select the PPG, BP, and Compliance menus to take photoplethysmograph, blood pressure, and compliance measurements by clicking the PPG, BP, and Compliance buttons. Next, you will be directed to the preparation page for the priority measurement of the patient's condition as shown in the following image:

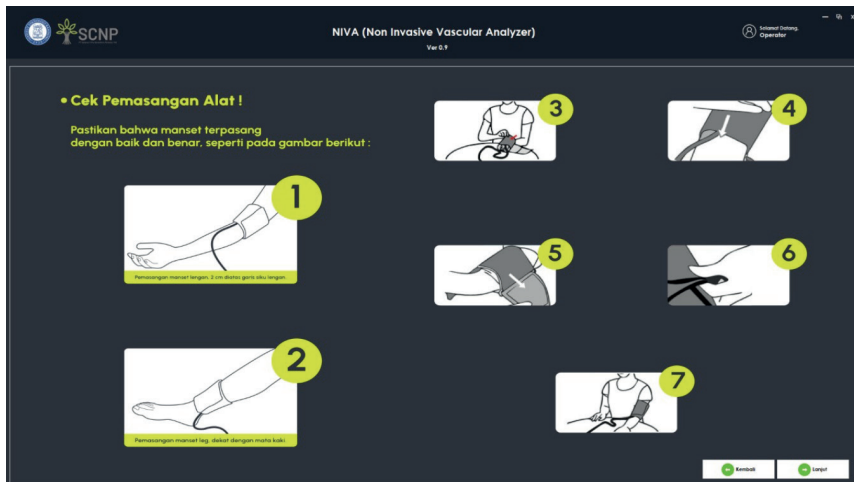
Then click the continue button and you will be directed to the measurement preparation page for the cuff/cuff installation check. Please pay close attention to how to install the cuff because it greatly affects the measurement results. The cuff/cuff installation check page is shown as follows:

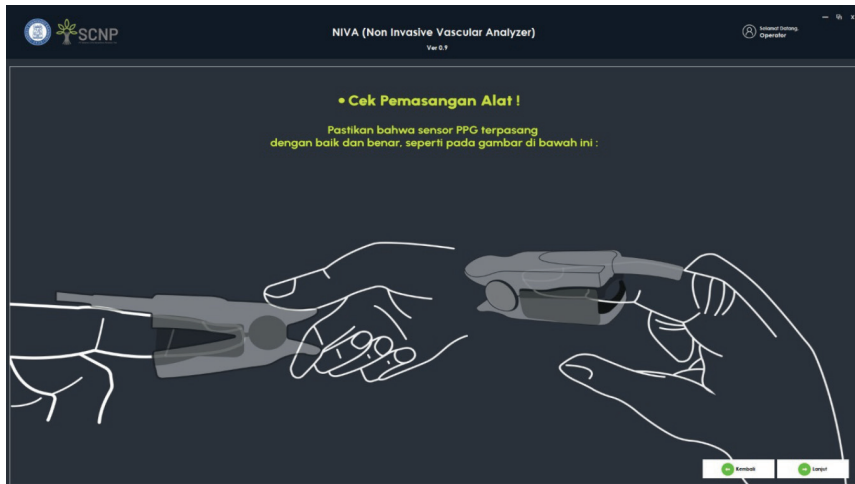Then click the continue button and you will be directed to the preparation page for the measurement of the photoplethysmograph (PPG) sensor installation check. Please pay close attention to how to install the photoplethysmograph (PPG) sensor because it greatly affects the measurement results. The photoplethysmograph (PPG) sensor installation check page is shown as follows:

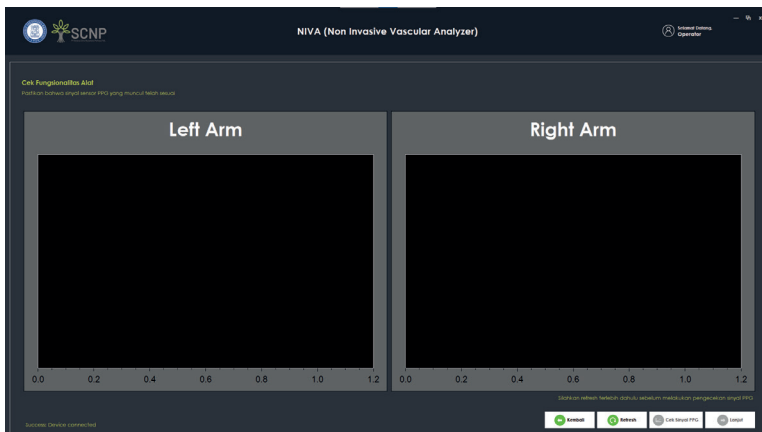
Then after clicking continue, you will be directed to the photoplethysmograph (PPG) sensor functionality checking page so that there are no errors in the data collection process.


If the signal from the photoplethysmograph (PPG) sensor does not appear, then it is necessary to re-connect with the photoplethysmograph (PPG) sensor device by clicking the Refresh button.

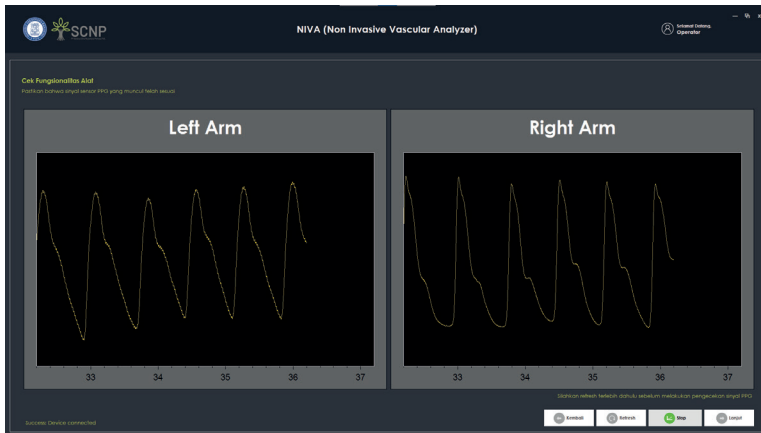Then click the PPG signal check button to see the signal wave read by the photoplethysmograph (PPG) sensor. The results of the photoplethysmograph (PPG) sensor signal reading will appear as shown below:

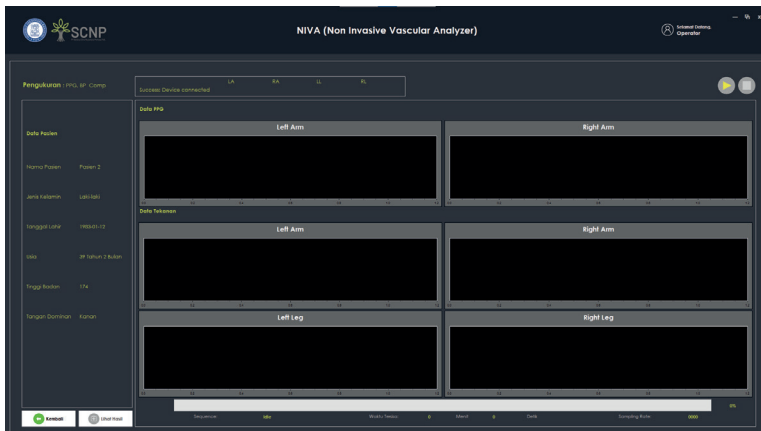
When it is sufficient to check the functionality of the photoplethysmograph (PPG) sensor, click stop to stop the functionality check process. Then click continue to continue the patient measurement process. After clicking the next button, you will be directed to the measurement page as follows:


Make sure that the cuff/cuff and photoplethysmograph (PPG) sensor are installed properly and that the patient's condition is priority. Then click the start button

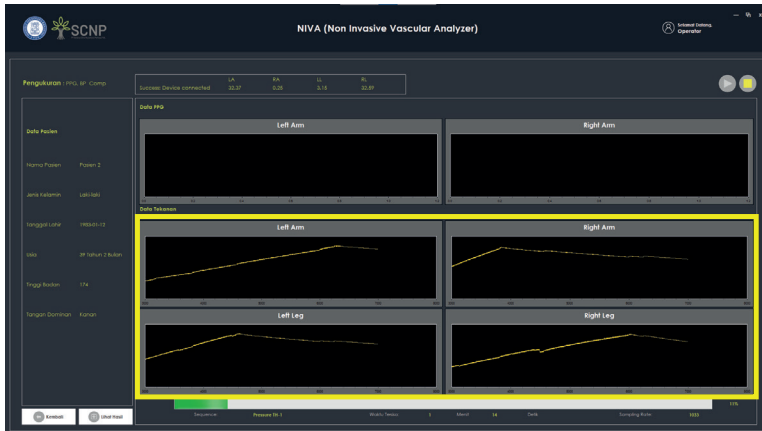Measurement is divided into 3 stages. The first step is measuring blood pressure by inflating the cuff/cuff on the right arm, left arm, right leg, and left leg as shown below:

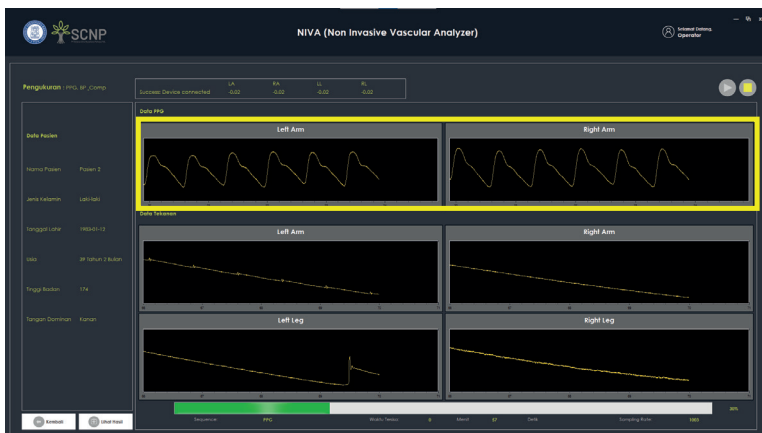
Then the next step is measurement with a photoplethysmograph (PPG) sensor on the right and left fingers as follows:

Then the last stage is compliance, which is the measurement on the non-dominant hand as follows:

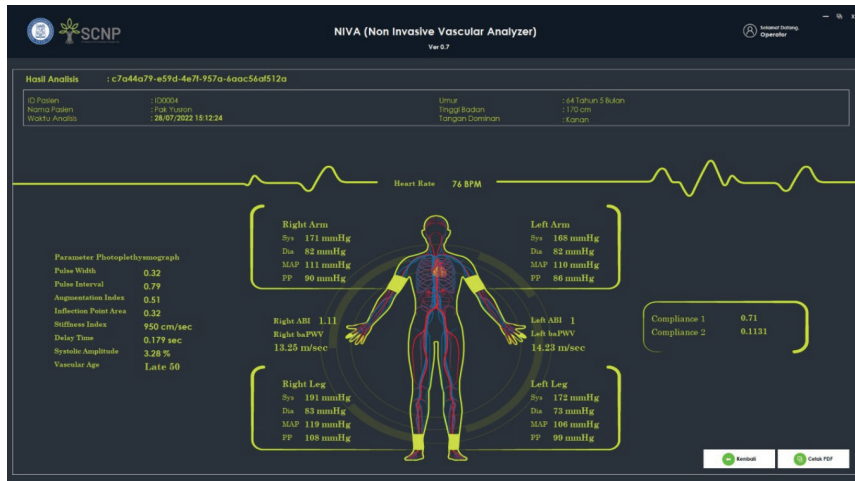After the measurement step is complete, the measurement results will appear as follows:


Click the print PDF button to get the measurement results in .pdf format.

#### 4.8 Patient Special Notes

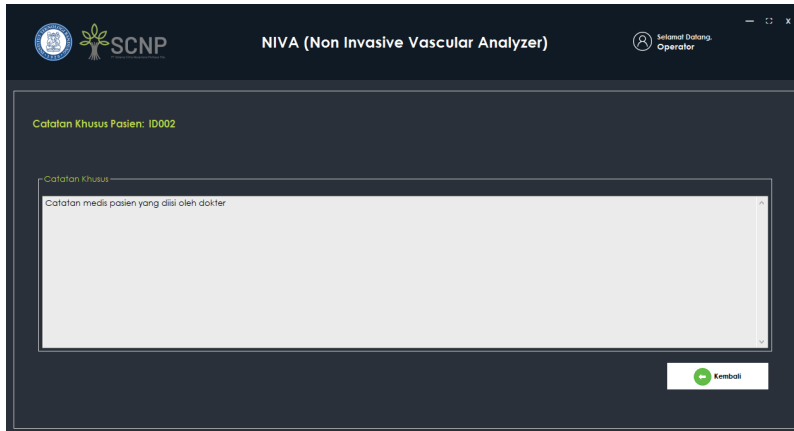On the main page, select the patient whose special notes will be viewed, then click the View Notes button, you will be directed to the patient's special notes page as follows:

#### 4.9 Update Patient Data

On the main page select the patient whose data will be updated then click on the update button it will be directed to the update patient data page as follows:

The screenshot shows the 'Perbarui Data Pasien: ID002' form in the NIVA software. The form contains the following fields and values:

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Buttons at the bottom right: Kembali, Selanjutnya

After being updated, click Next, you will be directed to the Update Special Notes page for patients as follows:

The screenshot shows the 'Perbarui Data Pasien: ID002' form in the NIVA software, specifically the 'Catatan Khusus' section. The text in the field is: 'Catatan medis pasien yang diisi oleh dokter'.

Buttons at the bottom right: Kembali, Perbarui

Then click the update button then the data will be updated according to the update.

#### 4.10 Patient Data Collection Log Data

On the main page, select the patient whose data collection log will be viewed. Then click the view data button, you will be directed to the patient retrieval data log page as follows:

**NIVA (Non Invasive Vascular Analyzer)**

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Then select the test data that you want to see the examination results by clicking the examination results button. Next, you will be directed to the inspection analysis log data page as follows:

**NIVA (Non Invasive Vascular Analyzer)**

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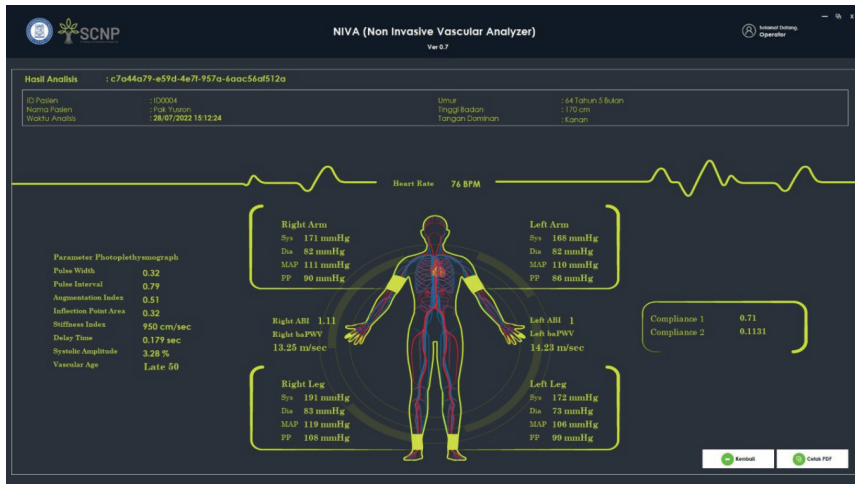
Kembali Hasil Analisis

Next, select the data you want to see the analysis results for then click the analysis results button, you will be directed to the analysis results page as follows:

#### 4.11 Export to Excel

On the main page click on the Export button (.xlsx) then you will be directed to the Export to Excel page as follows:

**NIVA (Non Invasive Vascular Analyzer)**  
Ver 0.7

Personal Data  
Operator

**Ekspor ke Excel**

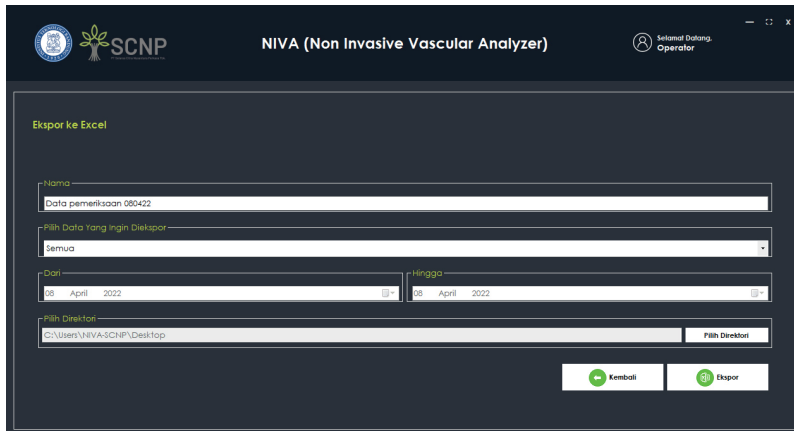
Nama:

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
Start Date: 08 April 2022  End Date: 08 April 2022

Pilih Direktori:


Then fill in the file name in the name textbox, then select the data you want to export, then select the data storage directory. Examples such as the following image:

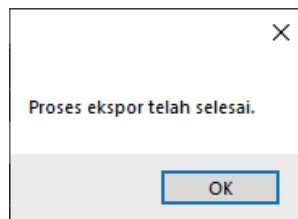Then click export, wait until a pop-up appears as follows:

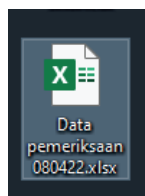Then check the .xlsx file in the previously specified directory. Examples such as the following image:

#### 4.12 Deleting Patient Data

On the main page, select the patient whose data you want to delete, then click the delete button, a pop-up will appear as shown below:

The screenshot shows the NIVA software interface. At the top, it says "NIVA (Non Invasive Vascular Analyzer)" and "Selamat Datang, Operator". Below this is a "Data Pasien" section with a search bar and a "Tambah Pasien" button. A table lists patient data:

ID	Name	Gender	MainHand	Created_At	Updated_At
ID0001	Pasien 1	Laki-laki	Kanan	07/04/2022 7:47	07/04/2022 7:12
ID0002	Pasien 2	Laki-laki	Kanan	08/04/2022 14:31	08/04/2022 14:31

A "Patient Deletion" dialog box is open, asking: "Are you sure you want to delete this patient: ID0001 ?" with "Yes" and "No" buttons.

At the bottom, there are buttons for "Ekspor(.xlsx)", "Pemeriksaan Baru", "Data Pemeriksaan", "Lihat Catatan", "Perbarui", and "Hapus". A red warning message reads: "HANYA UNTUK UJI COBA RUMAH SAKIT HARAPAN KITA, JAKARTA".


Click Ok then the selected patient data will be deleted in the NIVA system as shown below:

The screenshot shows the NIVA software interface after the deletion. The "Data Pasien" section now only contains one patient:

ID	Name	Gender	BirthDate	Height	MainHand	Created_At	Updated_At
ID0002	Pasien 2	Laki-laki	12/01/1983 14:30	174	Kanan	08/04/2022 14:31	08/04/2022 14:31

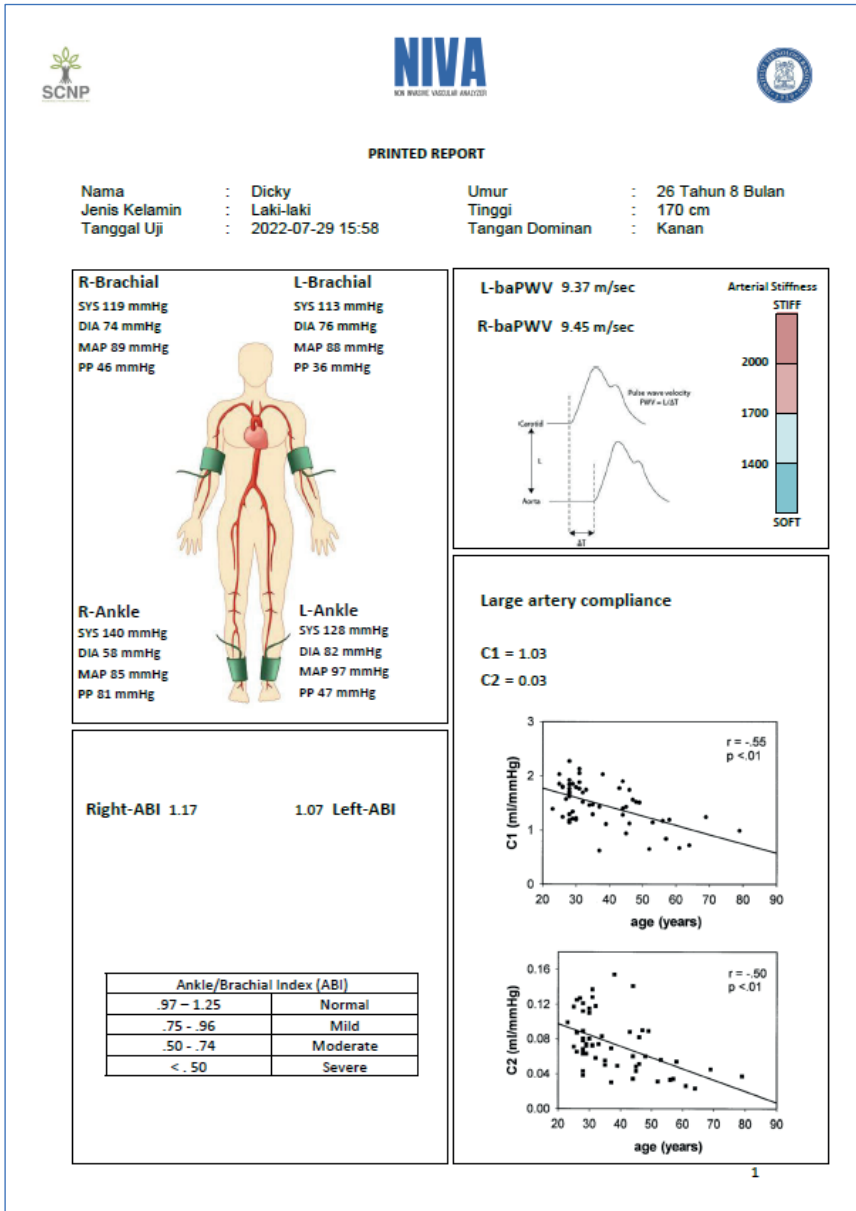The "Patient Deletion" dialog box is no longer present. The rest of the interface, including the bottom navigation buttons and the red warning message, remains the same.

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